




Remote Patient Manager  
Pairing your Garmin wristband




**Corin**


Responsible Innovation

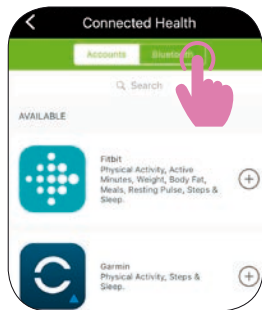! Please make sure that your smartphone has Bluetooth turned ON.




1

In the Corin RPM Patient app, go to  **Settings**  
↓  
 **Connected Health**  
↓  
 **Bluetooth**

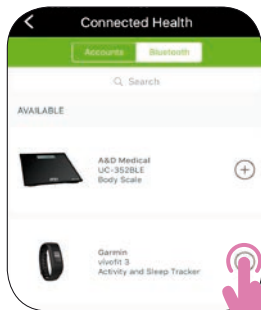

2

Scroll down to select **Garmin vivofit® 3** and tap on the 

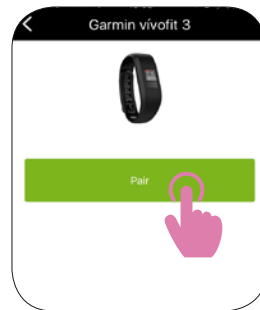


3

Tap on **“Pair”** in the app




4

Hold the pedometer button until **"PAIR"** appears on the LCD screen. **It may take a few minutes to complete pairing.**

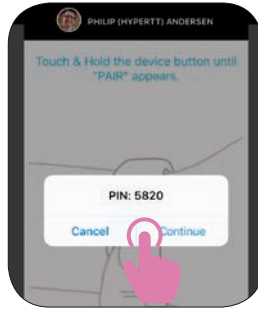

5

After completion a PIN number will be shown. Tap on **"Continue"** in the app and then **"Done"**.

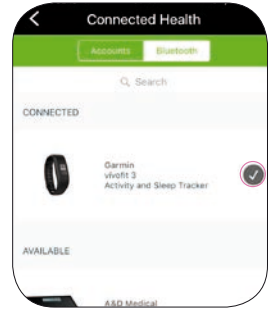

6

A checkmark will be shown next to the pedometer in the app. **You've successfully completed the pairing!**



If you have any questions regarding the pairing of your Garmin wristband please email us at [rpmsupport@coringroup.com](mailto:rpmsupport@coringroup.com)

**Corin**  
[www.coringroup.com](http://www.coringroup.com)